

Instructions for use

AF™ Baby File

[Cleaning, Disinfection and Sterilization]:

The product is provided in a non-sterile state; it is single use device.

It is required to be processed before each use by following Step-by-Step instructions, with steam sterilization at 126°C for 30 minutes.

Step-by-Step instructions:

- 1) Steam sterilization at 126°C for 30 minutes;
- 2) Rinsethe device with Normal saline (0.9% NaCl) for 10 seconds;
- 3) Dry the device with a dental air gun;



The first file system specially designed for primary teeth.

AFTM-HWire technology.

Triangular cross section design.

Improved resistance to cyclic fatigue.

Advanced tip processing & avoid ledge formation.

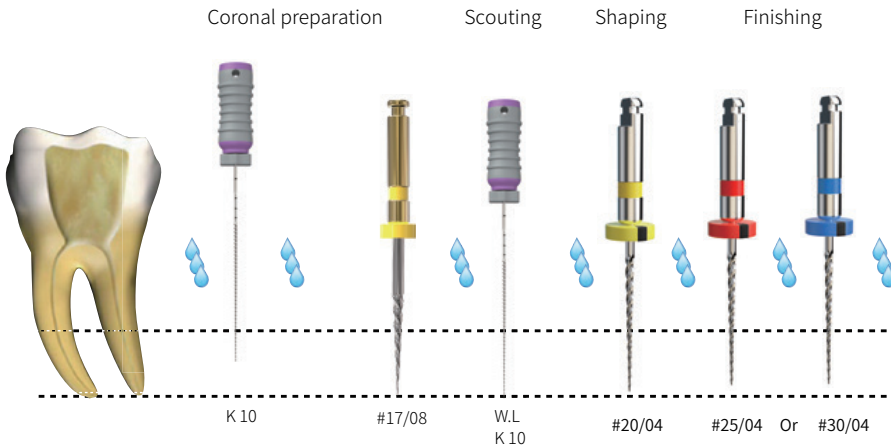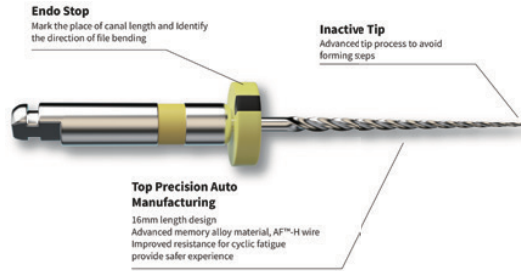

1. Negotiate the coronal 2/3 of the canal with K file 10.
2. Use Open File to prepare the coronal third and getting a straight line access.
3. Irrigate the canal.
4. With the use of apex locator,negotiate the canal with K file 10 in a watch-wind motion to full working length, to get a patent canal pathway.
5. Irrigate the canal.
6. Use file (#20/0.04) in a pecking motion to full working length.
7. Irrigate the canal.
8. Use file(#25/04) in a pecking motion to full working length.
9. Irrigate the canal.
10. You can stop at this file, or if the canal needs more preparation, you can proceed with file (#30/04) in a pecking motion.
11. Irrigate the canal.

SHANGHAI FANTA DENTAL MATERIALS CO.,LTD
 Room 1302,Building 2,No.6965 Pu Xing Rd.,Qingcun
 Town,Fengxian Dist,201414 Shanghai China
 +0086-2164033133
WWW.FANTA-DENTAL.COM



⚠ NI-TI Allergy is Prohibited