

Instructions for use

Max Cleaning

[Cleaning, Disinfection and Sterilization]:

The product is provided in a non-sterile state; it is single use device.

It is required to be processed before each use by following Step-by-Step instructions, with steam sterilization at 126°C for 30 minutes.

Step-by-Step instructions:


- 1) Steam sterilization at 126°C for 30 minutes;
- 2) Rinsethe device with Normal saline (0.9% NaCl) for 10 seconds;
- 3) Dry the device with a dental air gun;

- Minimal invasive file with maximum cleaning.
- Minimal stress applied on dentinal walls.
- Super elasticity with extreme flexibility.
- Maximum cleaning of the canal with smear layer removal.



Max 3

Max 1

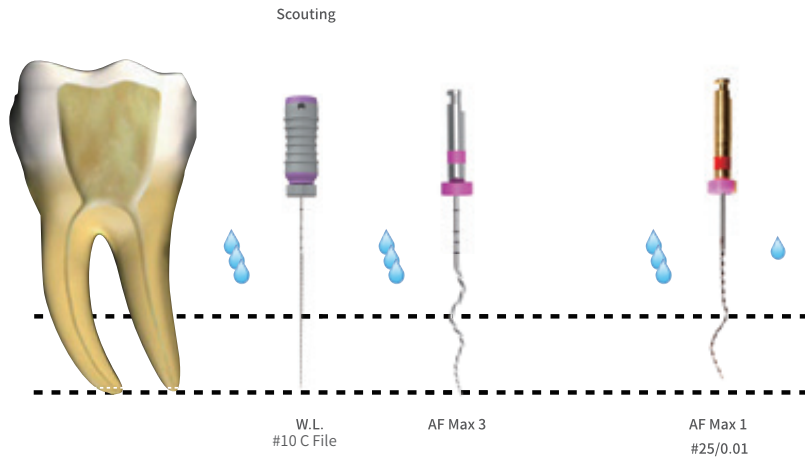

Scouting

W.L.
#10 C File

AF Max 3

AF Max 1
#25/0.01

1. Irrigate the canal.
2. With the use of apex locator, negotiate the canal with C File 10 in a watch-wind motion to full working length, to get a patent canal pathway.
3. Irrigate the canal.
4. Use Max 3 in a pecking motion (Pecking motion: in and out motion for a depth of 3 mm) till reaching the working length. The motion of the file is on rotation movement (speed 800RPM, Torque 1N).
5. Irrigate the canal.
6. To ensure better cleaning of the apical third and agitation of irrigation, use Max 1 with the same manner and settings mentioned in step 4.
7. Irrigate the canal.

SHANGHAI FANTA DENTAL MATERIALS CO.,LTD
Room 1302, Building 2, No.6965 Pu Xing Rd., Qingcun
Town, Fengxian Dist, 201414 Shanghai China
+0086-2164033133
WWW.FANTA-DENTAL.COM



⚠ NI-TI Allergy is Prohibited